



**WARNING! DEATH or SERIOUS INJURY can occur if manufacturer's instructions are not followed correctly.**


## Product Instructions

### E-Z-ON® Portable Wrist Restraints : Model #100WRP

- \* **Use** wrist restraints for added security when extra restraint is needed.
- \* **Deters** unwanted arm movements.
- \* **Clip** to the D-rings on the hip straps of the E-Z-ON VEST or may be sewn on, to prevent loss.
- \* **Each** restraint features a McDonald buckle closure with one-pull adjustment for quick securement.
- \* **Wrap** around wrist and secure with McDonald buckle.
- \* **Adjust** by pulling on webbing and secures restraints snug around wrist.
- \* **Back thread** webbing to lock in position.
- \* **Follow these instructions for proper use and care.**

Note: Sold as pair

E-Z-ON Wrist Restraints

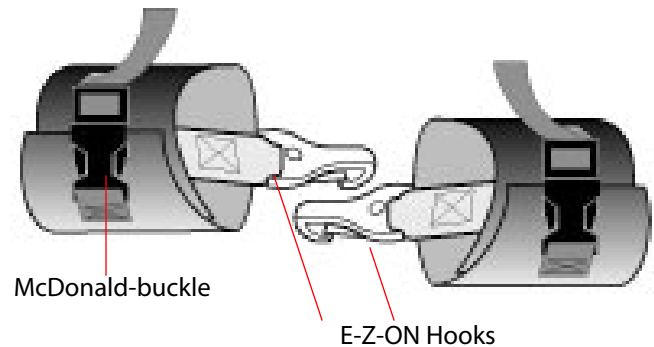

E-Z-ON® Products Inc. of Florida


605 Commerce Way West • Jupiter, FL 33458  
800-323-6598 • Fax 561-747-8779 • [www.ezonpro.com](http://www.ezonpro.com)

© 2005 E-Z-ON®

# Installation Instructions

## E-Z-ON Portable Wrist Restraints : Model #100WRP

1. **Wrap** one restraint around each wrist and secure McDonald-buckle.
2. **Position** corresponding McDonald-buckle on outside of wrist.
3. **Adjust** by pulling on extra webbing to secure restraints snug around wrist.
4. **Clip** E-Z-ON hooks to the D-rings on the hip straps of the E-Z-ON VEST.
5. **Secure** to **D-ring** of vest with hook facing inward.
6. **Tighten** each restraint to desired fit, back thread to lock.

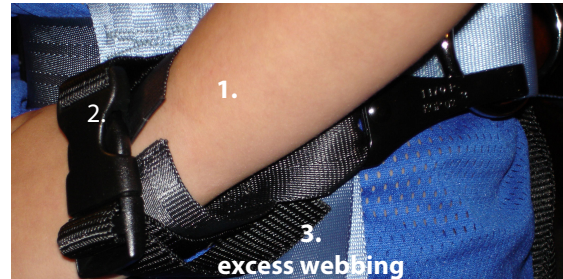

Hip D-Ring



E-Z-ON® Products Inc. of Florida

605 Commerce Way West • Jupiter, FL 33458  
800-323-6598 • Fax 561-747-8779 • [www.ezonpro.com](http://www.ezonpro.com)  
© 2005 E-Z-ON®