
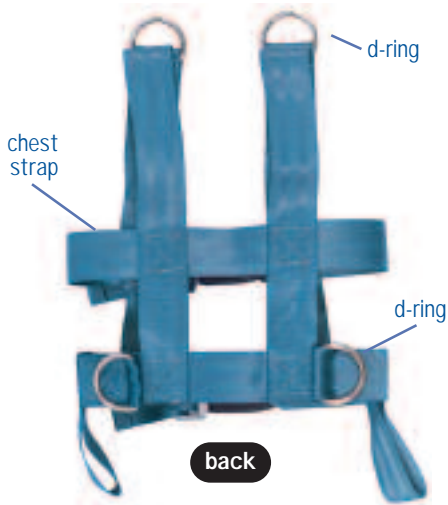


**! WARNING! DEATH or SERIOUS INJURY can occur if manufacturer's instructions are not followed correctly.**

# E-Z-ON® Model #101/102 Push Button Vest *for Family Vehicles*

- Meets FMVSS213.
- Use only for passengers 20-168 lbs.
- **Must use** in family vehicles with E-Z-ON® tether mount or floor mount.
- Use a vehicle belt to install this restraint with an E-Z-ON® Tether mount.
- **Replace** after a crash or 5 years from the date of manufacture.
- **Be sure** to register this restraint to be notified of a safety recall.

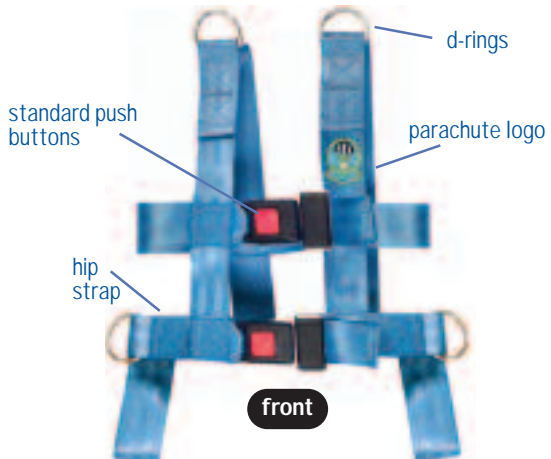

**E-Z-ON®**  
Products, Inc. of Florida

605 Commerce Way West • Jupiter, FL 33458

800-323-6598/561-747-6920 • Fax 561-747-8779 • [www.ezonpro.com](http://www.ezonpro.com)

© 2005 E-Z-ON®




# E-Z-ON® Model #101/102 Push Button Vest

## *Installation Guide for Family Vehicles*

1. Place the vest on the child with the parachute logo on the front.
2. Close and secure the push buttons
3. Pull webbing ends snug. Be sure the vest fits secure.



*If your vest has crotch straps follow below, otherwise go to step 8.*

4. Bring the crotch straps through the passenger's legs.
5. Thread each strap through the correct crotch buckle. Start through the rear of each top buckle-slot. Thread downward.
6. Adjust straps through each buckle to shorten or lengthen each crotch strap. Adjust the crotch straps snug, but not tight.
7. Back-thread web to lock webbing.

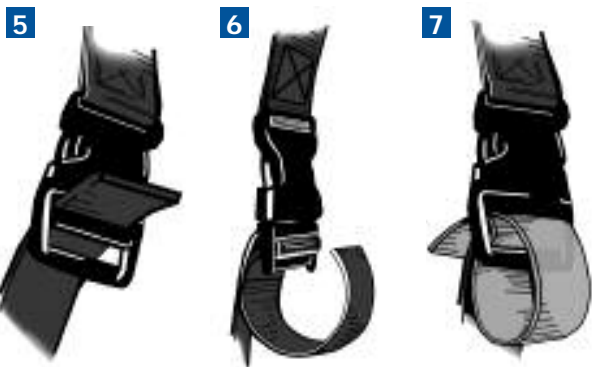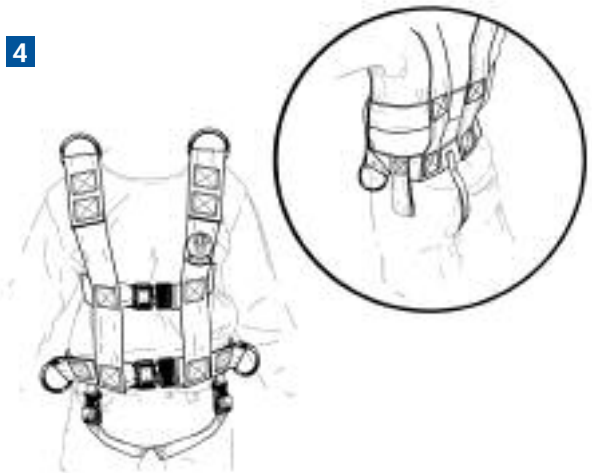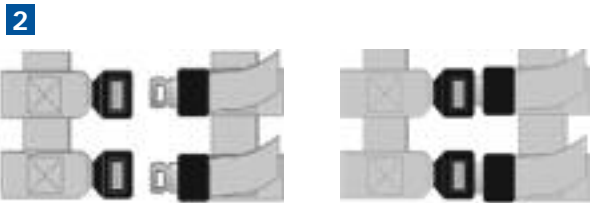

8. Keep hip straps low around the lap area.
9. Attach snap hooks to the d-rings on the shoulders of the vest.
10. Use both shoulder and hip strap d-rings to secure snap hooks on the E-Z-ON® floor mount, and wheelchair mount.
11. Use a vehicle belt and shoulder d-rings when using the E-Z-ON® tether mount.
  - a. thread a lap-only belt through the vest belt loops, buckle the belt, pull snug over the lap area.
  - b. lock a lap/shoulder belt with a locking clip, (see figure 11), thread belt with secured locking clip, through the vest belt loops, buckle the belt, pull snug over the lap area. Use the vest belt loops and the shoulder d-rings for the E-Z-ON® Tether mount only.
12. A booster can be used instead of crotch straps to secure. Use crotch straps only as a belt guide, do not use vest belt loops with booster.

8 &amp; 9


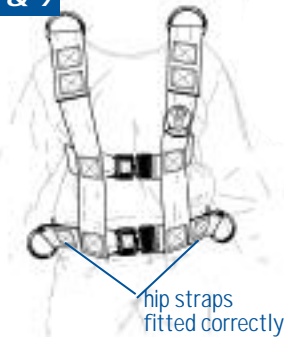
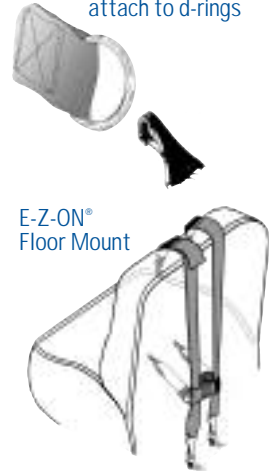
snap hooks  
attach to d-ringsE-Z-ON®  
Floor Mount

10

E-Z-ON®  
Wheelchair  
Mount

11

Locking clip

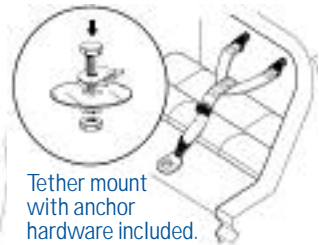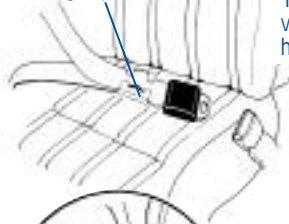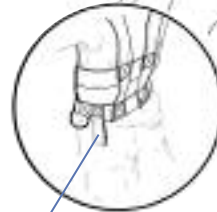Tether mount  
with anchor  
hardware included.belt loop for  
vehicle belt

12



*Be sure to register your E-Z-ON® Push Button Vest for Family Vehicles information on the back.*

## How to Register This Child Restraint

Child restraints could be recalled for safety reasons.

You **must** register this restraint in order to be notified of a recall.

**Fill out** the REGISTRATION CARD enclosed with your E-Z-ON® Vest, **send** your name address and the restraint model number and manufacturing date to the address below or **log on** to our website [www.ezonpro.com](http://www.ezonpro.com) and **fill out** your product information via our registration page.

For recall information, **call** the U.S. Government Auto Safety Hotline at 1 (800) 424-9393. In the Washington D.C. area call (202) 366-0123.



**E-Z-ON®**  
**Products, Inc. of Florida**

605 Commerce Way West • Jupiter, FL 33458  
800-323-6598/561-747-6920 • Fax 561-747-8779 • [www.ezonpro.com](http://www.ezonpro.com)  
© 2005 E-Z-ON®