

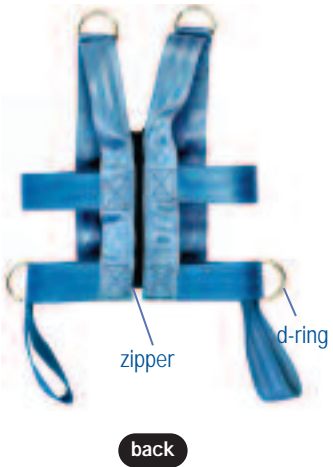


! WARNING! DEATH or SERIOUS INJURY can occur if manufacturer's instructions are not followed correctly.

**E-Z-ON®
Model
#101/102
Zipper Vest
for Family Vehicles
Owners Manual**

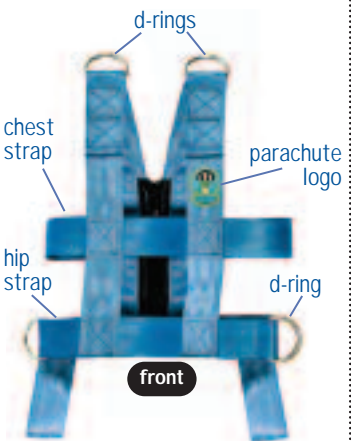

- 101 Zipper Vests XS-M come with crotch straps meeting NHTSA guidelines.
- Portable crotch or sewn straps are available to order for larger vests.
- **Do not** use a vehicle seatbelt with this vest. Use only E-Z-ON® vests with belt loops and a vehicle lap-belt if an E-Z-ON® tether mount is used in a vehicle.
- Meets FMVSS213.
- **Use** for a passenger 20-168 lbs. only.
- An E-Z-ON® tether mount, floor mount or wheelchair mount **is needed** to install this restraint correctly.
- **Replace** after a crash or 5 years from the date of manufacture.
- **Be sure** to register this restraint to be notified of a safety recall.

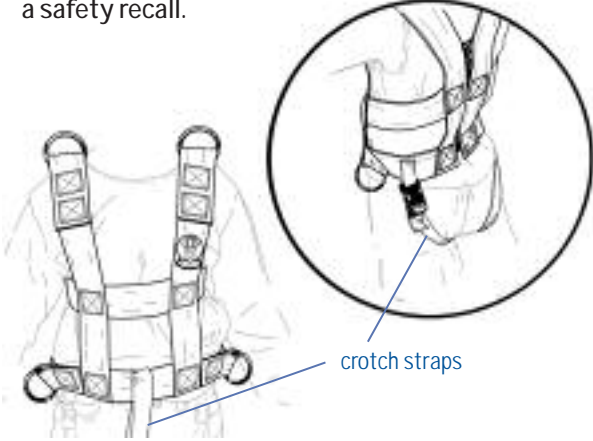

**E-Z-ON®
Products, Inc. of Florida**


605 Commerce Way West • Jupiter, FL 33458
800-323-6598/561-747-6920 • Fax 561-747-8779 • www.ezonpro.com
© 2005 E-Z-ON®

E-Z-ON® Model #101/102 Zipper Vest *for Family Vehicles* Owners Manual

1. Place the vest on the passenger with the parachute logo on the front.
2. Zip the vest completely. Be sure the vest fits secure. Keep the hip straps low around the lap area.

If your vest is XS-M or has a sewn in crotch strap follow the steps below. Otherwise go on to step 7.

3. Bring the crotch straps back through the passenger's legs.
4. Thread each strap through the correct crotch buckle. Start through the rear of each top buckle-slot. Thread downward.
5. Adjust straps through each buckle to shorten or lengthen each crotch strap. Adjust the crotch straps snug, but not tight.
6. Back-thread webbing to lock down.
7. Keep hip straps low around the lap area.

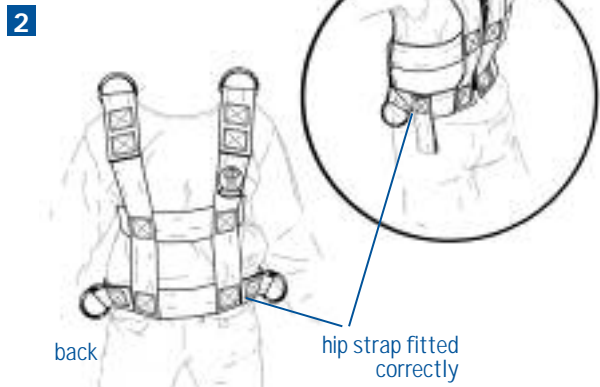

8. Use shoulder and hip d-rings to secure snap hooks for either the E-Z-ON® floor or wheelchair mount with this vest.

9. Follow E-Z-ON® wheelchair or floor mount instructions to secure snap hooks to the vest.

10. Use a vehicle belt and shoulder d-rings when using the E-Z-ON® tether mount.

a. thread a lap-only belt through the vest belt loops, buckle the belt, pull snug over the lap area.

b. lock a lap/shoulder belt with a locking clip, (see figure 11), thread belt with secured locking clip, through the vest belt loops, buckle the belt, pull snug over the lap area. Use the vest belt loops and the shoulder d-rings for the E-Z-ON® Tether mount only.

11. A booster can be used instead of crotch straps. Do not use vest loops with booster.

9



Snap hooks are secured to d-rings on the vest.

10





E-Z-ON® Wheelchair Mount



E-Z-ON® Floor Mount


11

Locking clip

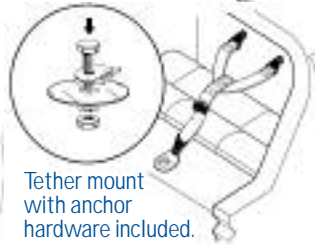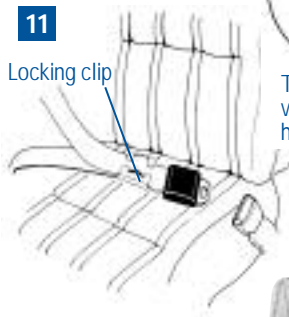

Tether mount with anchor hardware included.

E-Z-ON® hook attaches to hip d-rings on the vest

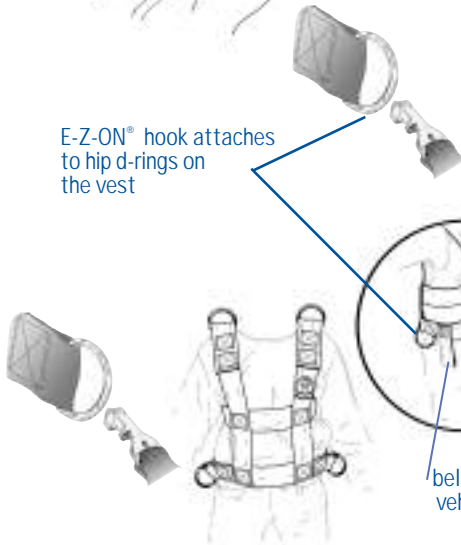

belt loop for vehicle belt

Be sure to register your E-Z-ON® Vest for family vehicles information on the back.

How to Register This Child Restraint

Child restraints could be recalled for safety reasons.

You **must** register this restraint in order to be notified of a recall.

Fill out the REGISTRATION CARD enclosed with your E-Z-ON® Vest, **send** your name address and the restraint model number and manufacturing date to the address below or **log on** to our website www.ezonpro.com and **fill out** your product information via our registration page.

For recall information, **call** the U.S. Government Auto Safety Hotline at 1 (800) 424-9393. In the Washington D.C. area call (202) 366-0123.



E-Z-ON®
Products, Inc. of Florida

605 Commerce Way West • Jupiter, FL 33458
800-323-6598/561-747-6920 • Fax 561-747-8779 • www.ezonpro.com
© 2005 E-Z-ON®