


**! WARNING! DEATH or SERIOUS INJURY can occur if manufacturer's instructions are not followed correctly.**

# E-Z-ON<sup>®</sup> Model #103Z Adjustable Vest *for Family Vehicles*

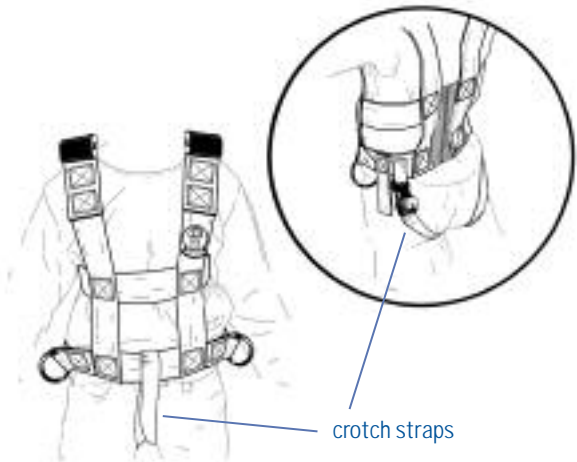

- 103Z Vests XXS-M come with crotch straps meeting NHTSA guidelines.
- Portable crotch or sewn straps are available to order for larger vests
- Meets FMVSS213.
- **Use** for a passenger 20-168 lbs. only.
- **Use** an E-Z-ON<sup>®</sup> tether mount, floor mount or wheelchair mount to install this restraint correctly.
- **Replace** after a crash or 5 years from the date of manufacture.
- **Be sure** to register this restraint to be notified of a safety recall.



**E-Z-ON<sup>®</sup>**  
**Products, Inc. of Florida**


605 Commerce Way West • Jupiter, FL 33458  
800-323-6598/561-747-6920 • Fax 561-747-8779 • [www.ezonpro.com](http://www.ezonpro.com)  
© 2005 E-Z-ON<sup>®</sup>

# E-Z-ON® Model #103Z Adjustable Vest *Installation Guide for Family Vehicles*

1. Place the vest on the passenger with the parachute logo on the front.
2. Choose one of the 3 zippers that fits the passenger snug. **Be sure** the vest fits secure. **Keep** the hip straps low around the lap area.

*If your vest is XXS-M or has a sewn in crotch strap follow the steps below. Otherwise go on to step 7.*

3. Bring the crotch straps back through the passenger's legs.
4. Thread each strap through the correct crotch buckle. **Start** through the rear of each top buckle-slot. **Thread** downward.
5. Adjust straps through each buckle to shorten or lengthen each crotch strap. **Adjust** the crotch straps snug, but **not** tight.
6. Back-thread webbing to lock webbing.

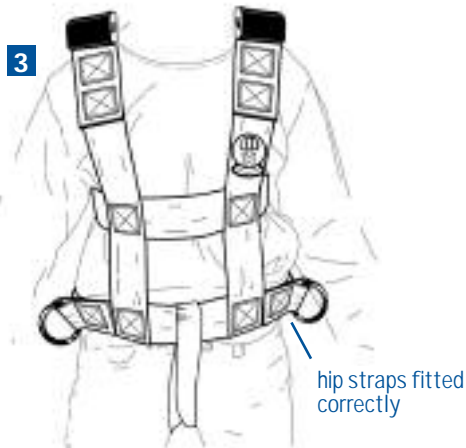

7. **Secure** the shoulder straps with the shoulder adjusters. **Pull** webbing ends through adjusters to create a snug fit. **Remove** any webbing slack. **Do not** pull too tight. This will pull the vest up. **Back-thread** the webbing ends through the barslides to lock.

8. **Keep** hip straps low around the lap area.

9. **Use** both hip d-rings and the tether hook slots for the E-Z-ON® wheel chair mount and floor mount **only**.


10. **Use** a vehicle belt and tether slots to secure the E-Z-ON® tether mount:  
**a. thread** a lap-only belt through the vest belt loops, **buckle** the belt, pull snug over the lap area.

11. **Lock** a lap/shoulder belt with a locking clip, (see figure 11) **thread** belt with secured locking clip, through the vest belt loops, **buckle** the belt, **pull** snug over the lap area.

12. A booster can be used instead of crotch straps to secure. Use crotch straps only as a belt guide, do not use vest belt loops with booster.



**Read** both the E-Z-ON® mount and vest instructions carefully.

*Be sure to register your E-Z-ON® Model 103Z Vest for vehicles information on the back.*

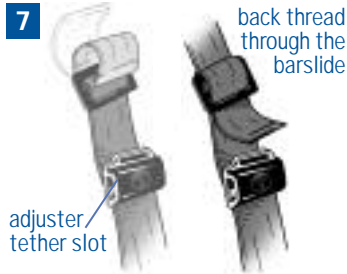

snap hooks attach to the tether slots on each shoulder of the vest

snap hooks ALSO attach to d-rings located on the hip straps on the vest

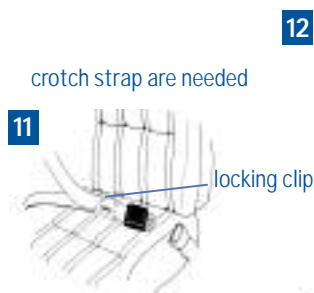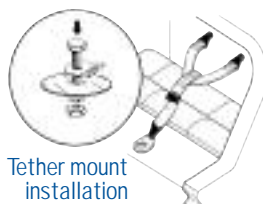

crotch strap are needed

vest with a booster no crotch straps are needed

## How to Register This Child Restraint

Child restraints could be recalled for safety reasons.

You **must** register this restraint in order to be notified of a recall.

**Fill out** the REGISTRATION CARD enclosed with your E-Z-ON® Vest, **send** your name address and the restraint model number and manufacturing date to the address below or **log on** to our website [www.ezonpro.com](http://www.ezonpro.com) and **fill out** your product information via our registration page.

For recall information, **call** the U.S. Government Auto Safety Hotline at 1 (800) 424-9393. In the Washington D.C. area call (202) 366-0123.



**E-Z-ON®**  
**Products, Inc. of Florida**

605 Commerce Way West • Jupiter, FL 33458  
800-323-6598/561-747-6920 • Fax 561-747-8779 • [www.ezonpro.com](http://www.ezonpro.com)  
© 2005 E-Z-ON®