



WARNING! DEATH or SERIOUS INJURY can occur if manufacturer's instructions are not followed correctly.


E-Z-ON[®]

Model #101M2

Modified Push Button Vest

Installation Instructions

- * Meets FMVSS213.
- * **Fits Children 5-12 yrs old**
20-100 lbs. who need to lie down.
- * **Passenger** must fit on the vehicle seat lengthwise.
- * **Thread 2** rear belts through sides of the vest and the extremity belt. The extremity belt is used to secure casted or uncashed legs.
- * **Transport** in **supine or prone** positioning. Vest must be snug around the child.
- * **Use** for body casts, hip spica casts, braces or any other configuration where the child must be transported lying down.
- * Great for **Sleeping Children** too.
- * **Replace** after a crash or 5 years from date of manufacture.
- * **Register** in case of a recall.

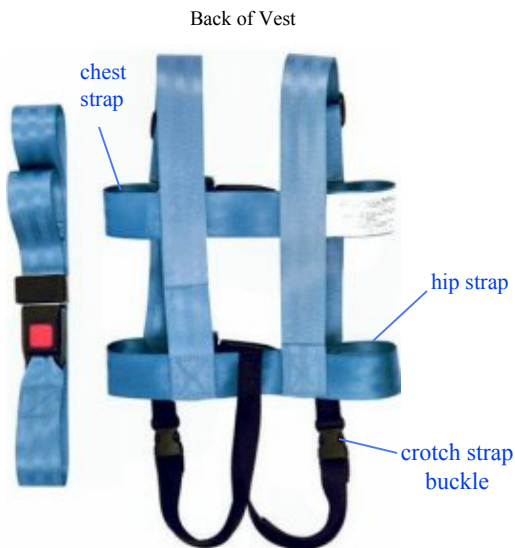

E-Z-ON[®] Products Inc. of Florida

605 Commerce Way West • Jupiter, FL 33458
800-323-6598 • Fax 561-747-8779 • www.ezonpro.com

© 2005 E-Z-ON[®]






E-Z-ON[®]

Model #101M2

Modified Push Button Vest

Instructions: Securing the vest on the Child.

1. Place the vest on the child with the parachute logo on the front.
2. Close and secure the push buttons. Pull webbing snug
3. Adjust fit, pull loose end of shoulder strap through barslide adjust shoulder straps so the vest is positioned over the pelvic bone.
4. **Hold** the barslide with one hand while pulling the loose end of shoulder strap to shorten or lengthen the strap & then back thread to lock. Bring the Crotch Straps through the passenger's legs and secure.
 - 5a. Thread each strap through the correct crotch buckle. Start through the rear of each top buckle-slot. Thread Downward.
 - 5b. **Adjust** straps through each buckle to shorten or lengthen the crotch strap. Adjust the crotch straps snug, but not tight.
 - 5c. **Back-Thread** the webbing through the top buckle to lock webbing. The crotch straps help keep the Modified vest correctly positioned over the pelvic bone during use.

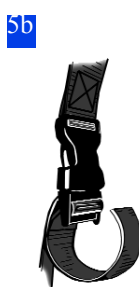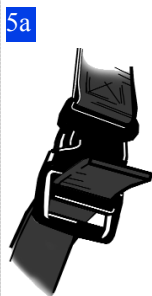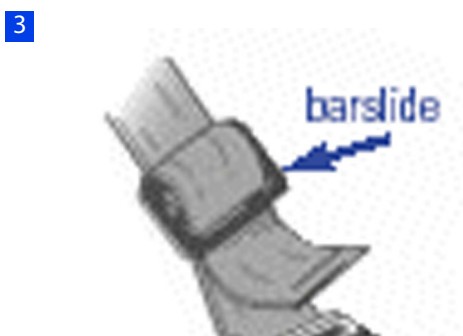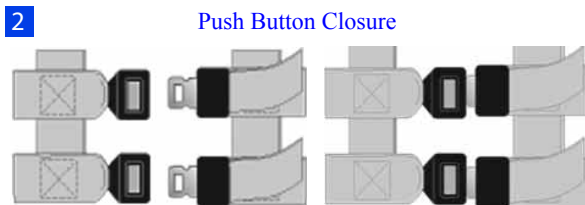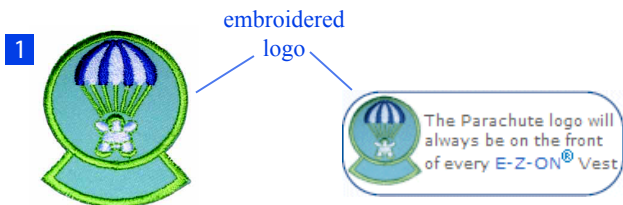

Securing the Child to the Seat.

6. Lay the child on the **rear seat** of the vehicle, placing the child in the center of the seat, away from the doors (or walls) of the vehicle.

a. Using the **existing seat belts**, place one seat belt through the chest strap of the vest and secure the seat belt.

b. Thread the other seat belt through the lower hip strap of the vest and secure the belt.

c. **Secure** both seat belts as tightly as possible.

7. The **extremity** belt provided with the E-Z-ON 101M2 Modified Vest should be used as follows:

a. If **one** leg is casted, be sure that the cast is against the seat back. Place the extremity belt around the leg with the **cast** at the **knee area**, and secure with the same seat belt as the **hip** strap of the vest.

b. If **both** legs are casted, place the extremity belt around **both casts** at the **knee area**, and secure with the same seat belt as the hip strap of the vest.



c. If **neither** leg is casted, place the extremity belt around **both legs** at the **knee area** and secure with the same seat belt as the **hip** strap of the vest.

8. The Modified E-Z-ON Vest can be secured using **lap type** seat belts or **shoulder/lap** belts. If shoulder/lap belts are used, you may use a **locking clip** to lock the lap-portion of the belt.

9. **Protect the Head and Neck!**

a. Place **pillows** behind the child's **head & neck area** for comfort and protection in case of a crash.

b. If the child needs to ride on their **side**, secure the vest the same way, but place a **pillow** behind the child's **head** for comfort and support.

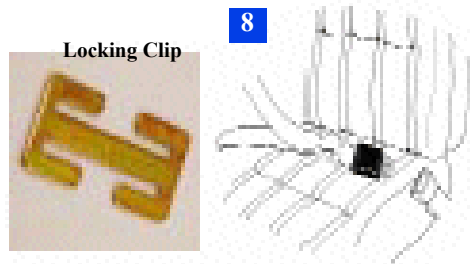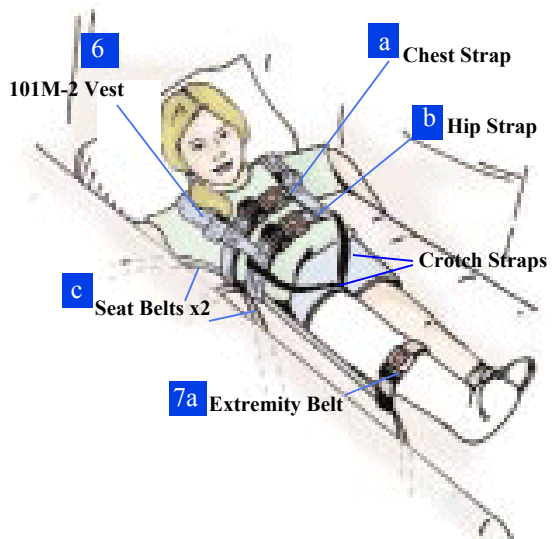

Great for Long trips...for all Children!
Now a child can nap comfortably in the car,
and parents can have peace of mind
knowing that their child is safely secured to
the vehicles seat belts.

How to Register This Child Restraint

Child restraints could be recalled for safety reasons.

You must register this restraint in order to be notified of recall.

Fill out the **REGISTRATION CARD** enclosed with your E-Z-ON™ Vest, send your name address and the restraint model number and manufacturing date to the address below or log on to our web site at **www.ezonpro.com**. and fill out your product information via our registration page.

For recall information, call the U.S. Government Auto Safety Hotline at

1 (800) 424-9393. In the Washington D.C. area call (202) 366-0123.



E-Z-ON® Products Inc. of Florida

605 Commerce Way West • Jupiter, FL 33458
800-323-6598 • Fax 561-747-8779 • www.ezonpro.com

© 2005 E-Z-ON®